Nr referencyjny Olympus: QIL FY25-EMEA-30-FY24-OSTA-06-1 Soltive Laser System

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Temat: System laserowy OLYMPUS Soltive™ SuperPulsed

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October 17, 2024

SOLTIVE™ Laser System Software Revisions

PN0015551 SOLTIVE Laser Instructions for Use

Your Soltive™ laser software and Instructions for Use (IFU) have been updated. Through this software upgrade, the Preset Treatment Parameters on your Soltive™ laser have been updated.

For a full printed copy of the Instructions for Use, please contact your local Olympus representative.

What has Changed:

1. On page 15 of the IFU, the NOTE of Treatment Parameters and Instructions has been updated as below:

NOTE

The SOLTIVE Laser System provides preset laser settings options for lithotripsy, soft tissue, and BPH procedures. However, individual treatment should be based on clinical training, clinical observation of laser-tissue interaction, and appropriate clinical endpoints. Always start with low settings and then increase them progressively to achieve the desired effect on targeted tissue. Olympus does not make recommendations regarding the practice of medicine.

2. The SOLTIVE Laser System provides pre-loaded laser settings options for Lithotripsy, Soft Tissue and Benign Prostatic Hyperplasia (BPH). The pre-loaded laser settings for Lithotripsy have been updated to have three options: Dusting, Bladder Stone, Ureteral Stone. Figure 31 Treatment Settings on page 35 of the IFU has been updated as follows:

BEFORE

Figure 31 Treatment Settings

AFTER

Figure 31 Procedure Presets

3. The preset values of Lithotripsy treatment parameters and Manual parameters are updated as shown in the below Table (page 50). Also, WARNING on page 49 has been updated.

WARNING

The use of laser energy that is too high may result in injury to the patient. Injuries include but are not limited to: possible renal impairment or tissue injury (blanching of tissue, bleeding, mucosal abrasion, perforation, and/or stenosis/stricture). Individual treatment should be based on clinical training, clinical observation of laser-tissue interaction, and appropriate clinical endpoints.

Please find updated instruction for Preset Treatment Parameters as below.

Table 4 Lithotripsy Treatment Preset Parameters

Lithotripsy		
Dusting	Bladder Stone	Ureteral Stone
Left Pedal Name: Dusting	Left Pedal Name: Bladder Stone	Left Pedal Name: Soft Stone
Energy: 0.2 J	Energy: 2 J	Energy: 0.5 J
Frequency: 30 Hz	Frequency: 9 Hz	Frequency: 8 Hz
Pulse Width: 1	Pulse Width: 1	Pulse Width: 1
Right Pedal Name: Fragmenting	Right Pedal Name: Hemostasis	Right Pedal Name: Hard Stone
Energy: 0.8 J	Energy: 1 J	Energy: 0.8 J
Frequency: 7 Hz	Frequency: 20 Hz	Frequency: 5 Hz
Pulse Width: 1	Pulse Width: 3	Pulse Width: 1

Table 7 Manual Parameters

Manual
Left & Right Pedals
Energy: 0.1 J
Frequency: 20 Hz
Pulse Width: 1

4. Additional Power Cord catalog numbers have been added in Table 11 SOLTIVE Accessories on page 52.

Table 11 SOLTIVE Accessories (1 Each)

Ref	Description
TFL-AFSWL	TFL Laser Wireless Footswitch
TFL-AFSW	TFL-Laser Footswitch — Wired
TFL-APCUS	SOLTIVE Laser Type B Standard Power Cord
TFL-APCEU	SOLTIVE Laser Type E/F Standard Power Cord
TFL-APC-G	SOLTIVE Laser Type G Standard Power Cord
TFL-APC-J	SOLTIVE Laser Type J Standard Power Cord
TFL-APC-L	SOLTIVE Laser Type L Standard Power Cord
TFL-APC-DK-HG	SOLTIVE Laser Denmark Power Cord - Hospital Grade
TFL-APC-M	SOLTIVE Laser Type M Standard Power Cord
TFL-APC-H	SOLTIVE Laser Type H Standard Power Cord
TFL-APC-I	SOLTIVE Laser Type I Standard Power Cord
TFL-APC-K	SOLTIVE Laser Type K Standard Power Cord
TFL-APC-O	SOLTIVE Laser Type O Standard Power Cord
TFL-AFC	TFL Fiber Cleaver
TFL-AFS150	TFL Fiber Stripper, 150 Micron, Autoclavable

5. The instruction for changing the batteries of wireless footswitch on page 56 has been updated as below.

Replace the batteries in the wireless footswitch with high quality AA Alkaline batteries. Never mix manufacturers when replacing batteries. Replace both batteries at the same time; do not mix old and new batteries. Old batteries must be discarded as regulated; do not throw them in a trash bin.